


ICS-96 to Patient Cable Adapter Instructions for Use

PN 7905

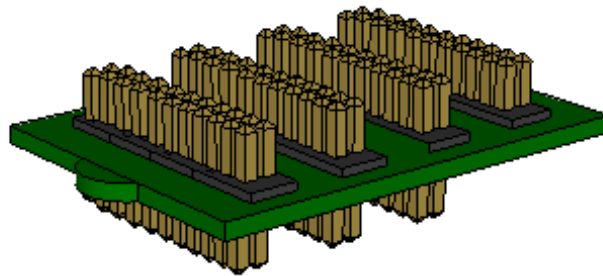

Blackrock Microsystems, Inc.

391 Chipeta Way, Suite G

Salt Lake City, UT 84108

Tel: (866) 806-3692

www.blackrockmicro.com

ICS-96 to Patient Cable Adapter – Instructions for Use

1 Hardware

1.1 ICS-96 to Patient Cable Adapter Pin-Out

The ICS-96 Adapter provides an interface between the Blackrock Patient Cable and third party devices. Third party devices connect to the ICS-96 to Patient Cable Adapter through the Samtec FTSH-118-04-L-DV connectors. The pin-out for this adapter is provided in the sections below.


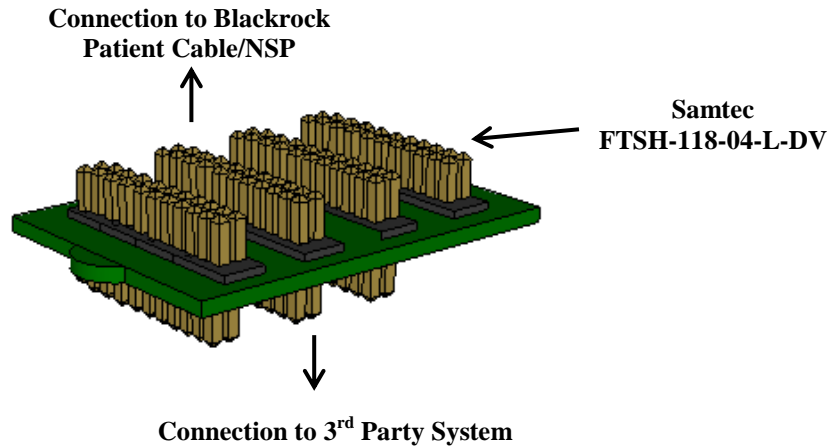



Figure 1: Quick Switch to Patient Cable Adapter

1.2 Top Layer (4 Connectors)

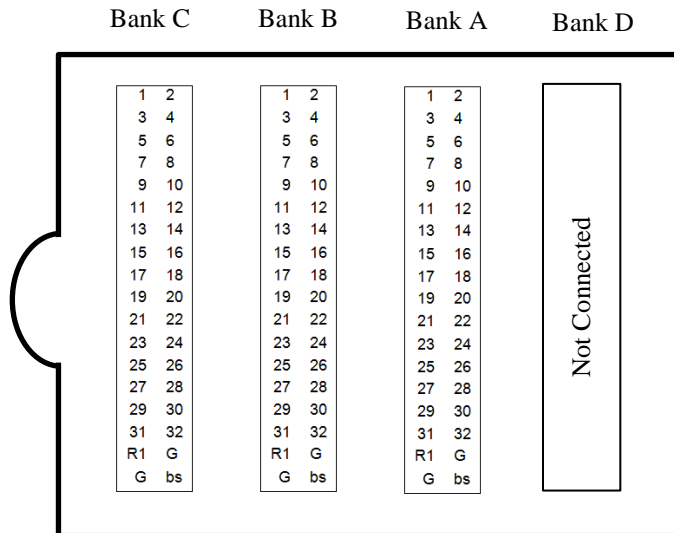


ICS-96 to Patient Cable Adapter – Instructions for Use

1.3 Bottom Layer (3 Connectors)

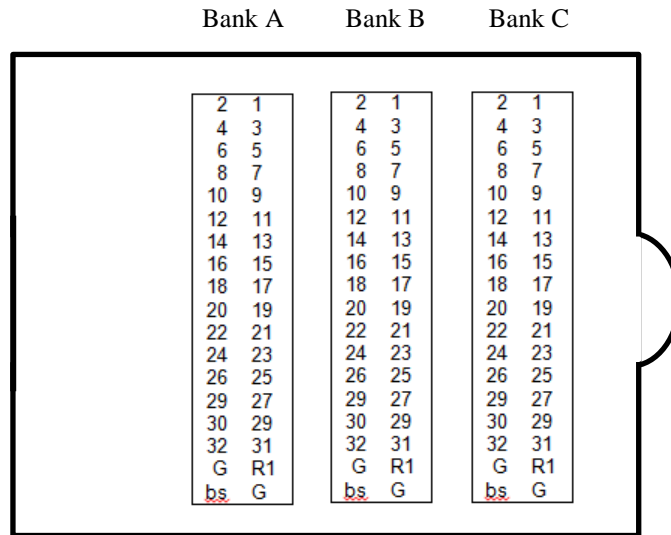

2 Warnings

	IEC60101-0102	Danger of Electrostatic Discharge (ESD)
--	---------------	---

- In order to reduce the likelihood of ESD, use antistatic or electrostatic discharge (ESD) safe gloves when connecting the ICS-96 to Patient Cable Adapter.

3 Warranty

Blackrock Microsystems, Inc. warrants that its products are free from defects in materials and manufacturing for a period of one year from the date of shipment. Blackrock will, at its option, repair or replace any product that does not comply with this warranty. This warranty is voided by:

1. Any modification or attempted modification to the product done by anyone other than an authorized Blackrock employee
2. Any abuse, negligent handling or misapplication of the product.

This constitutes the sole warranty made by Blackrock, Inc. There are no other warranties, expressed or implied, which extend beyond those described herein or to anyone other than the original purchaser, including the implied warranties of merchantability and fitness for a particular purpose. In no event shall Blackrock Microsystems, Inc. be liable for any incidental or consequential damages, or for the infringement of any patent rights or third party rights due to the use of its products.

3.1 Return Merchandise Authorization (RMA)

ICS-96 to Patient Cable Adapter – Instructions for Use

In the unlikely event that your adaptor needs to be returned to Blackrock for repair or maintenance, do not send any equipment back without a Return Merchandise Authorization Number. An RMA number will be issued to you by a Blackrock representative. If you need to obtain an RMA number, you may contact a product support representative at (801) 582-5533 or toll free at (866) 806-3692.

Once an RMA number has been issued, it is important to safely pack the returned item for shipping back to Blackrock. It is preferred that you save the original boxes and packing materials that your system arrived in for return shipment. Please address the package as follows:

Blackrock Microsystems, Inc.
ATTN: RMA#
391 Chipeta Way, Suite G
Salt Lake City, UT 84108 USA
Tel: (801) 582-5533